




**100056 Front Drawer Service Part for
Brewer Access Exam Table
Brewer Basic Exam Table
Brewer Access High-Low Table**

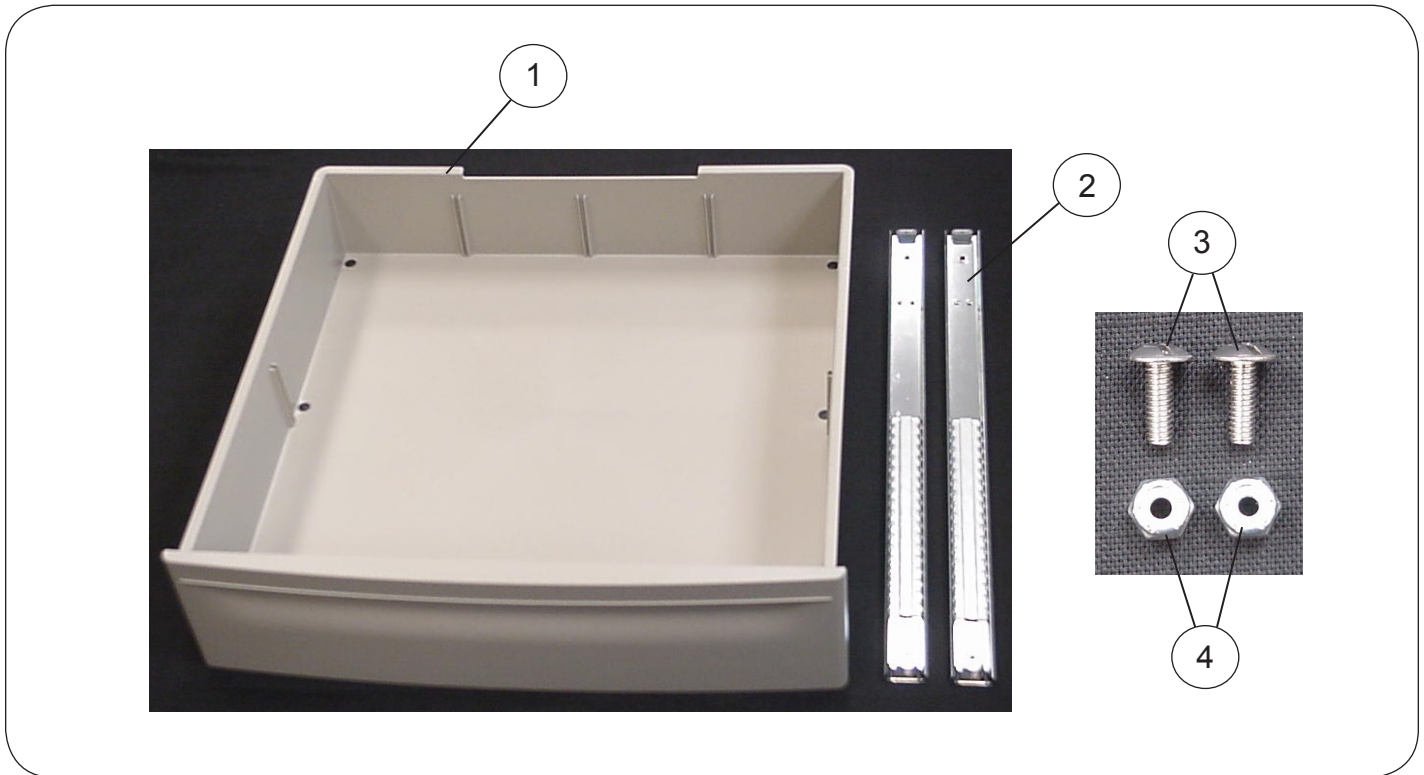

Item No.	Part No.	Description	Qty
1	--	Drawer	1
2	--	Drawer Slide	2
3	99107	#8 x 1/2" Truss Head Screw	2
4	99108	#8 Hex Locking Nut	2

The Brewer Company
13901 Main Street
Menomonee Falls, WI 53051
P 1.888.Brewer.1
F 262.251.1786
www.brewercompany.com




100056 Front Drawer Service Part for Brewer Access Exam Table Brewer Basic Exam Table Brewer Access High-Low Table

Tools Required:

- 11/32" Open End or Box Wrench
- Short Flat Blade Screwdriver
- Hammer with Small Cold Chisel or Drill with 3/16" bit

Drawer Removal:

1. Remove the faulty drawer by pulling both locking tabs to the left while pulling the drawer outward.

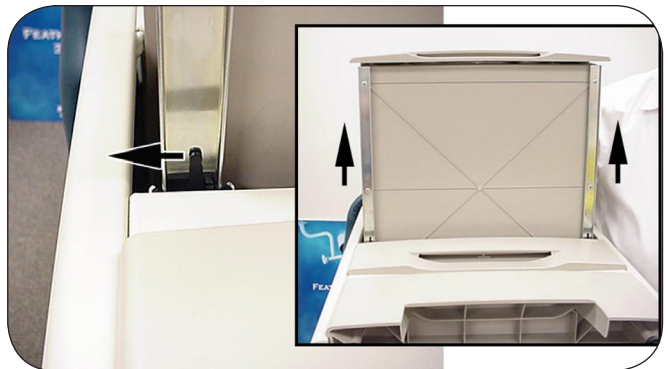

2. Using a cold chisel and hammer, remove the head of the rivet. You may also use a drill with a 3/16" bit. Repeat both sides.
3. Remove old drawer slides and discard.





OR



Drawer Install:

1. Locate the hook on the bottom side of the new drawer slide and the notch inside the drawer opening in the cabinet. Insert the hook on the new drawer slide into the notch in the cabinet.
2. Align the mounting hole in the front of the drawer slide with the hole in the front ledge of the drawer opening in the cabinet. Insert a #8 x 1/2" truss head screw and secure with #8 hex locking nut. Repeat for other side of drawer.

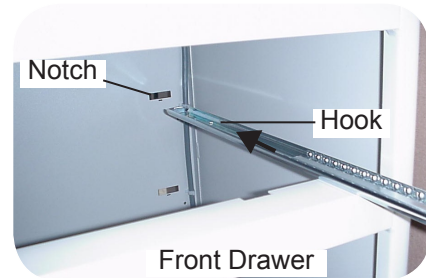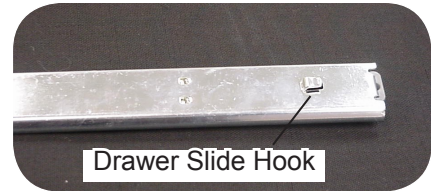

3. Before replacing the drawer, slide the glide track all the way forward inside the table. Align the drawer guides on both sides, pushing the drawer in while lifting up slightly.

