

Item Number	Part Number	Description	Quantity
1	100166	Storage Compartment Lid	1
2	100351	1/4" x 3/4" Carriage Bolt	3
3	000623	1/4" Hex Nut	3
4	99881	#10 x 3/4" Carriage Bolt	2
5	98474	#10 Hex Nut	2
6	100549	Bumper	2


The Brewer Company
 13901 Main Street
 Menomonee Falls, WI 53051
 P 1.888.Brewer.1
 F 262.251.1786
www.brewercompany.com

Tools Required:

- 3/8" Hex Socket Wrench
- 7/16" Hex Socket Wrench

Storage Compartment Lid Replacement:

1. Raise storage compartment lid to a vertical position.
2. Remove the top (2) #10 x 3/4" carriage bolt and hex nuts that secure the lid support arm to the storage compartment lid.
3. Remove the (3) 1/4" x 3/4" carriage bolt and hex nuts that secure the storage compartment lid to the hinge. Discard old lid.
4. Replace with new storage compartment lid, securing with new hardware provided.

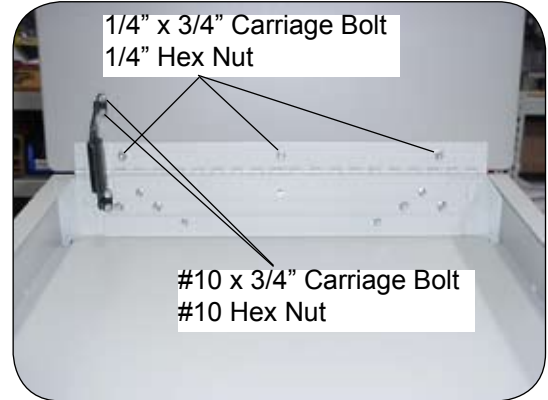

5. Position bumpers on the inside back corner surface of the lid so the lid rests on the bumpers when fully closed.

