


⚠ DANGER

Always disconnect the power cord from the wall outlet before performing service on the table to prevent electrical shock. Failure to do so could result in severe personal injury or death.

⚠ WARNING

Refer to the Operator's Manual supplied with the unit for complete instructions on operation. Failure to do so could result in personal injury.

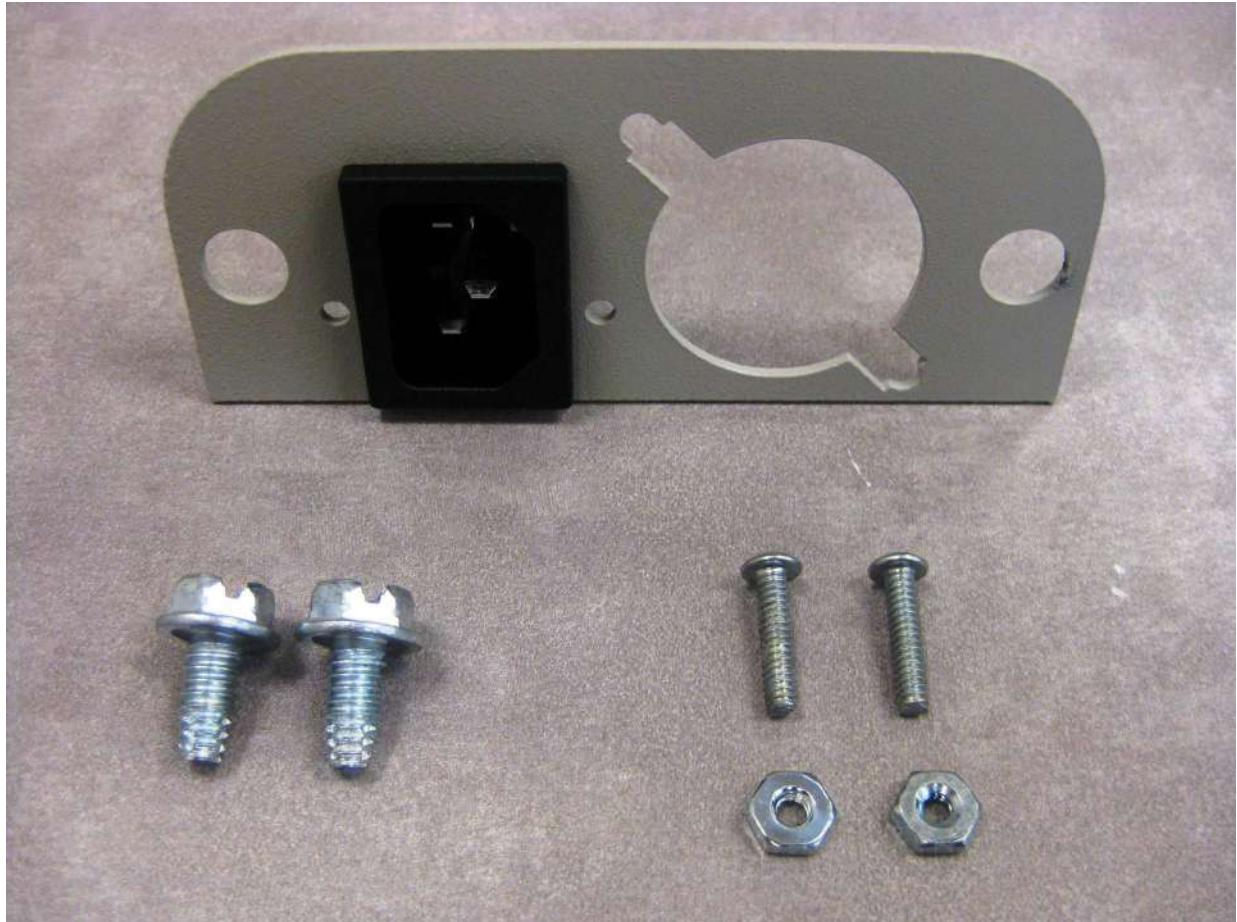

Description	Qty.
Electrical Plate	1
Power Inlet	1
Screw, 1/4-20	2
Screw, 6-32	2
Nut, 6-32	2

The Brewer Company, LLC
N88 W13901 Main Street, Suite 100
Menomonee Falls, WI 53051
P 1.888.BREWER.1
F 1.262.251.2332
www.brewercompany.com

TOOLS REQUIRED:

3/8" Nut Driver

5/64" Allen Wrench or Phillips Screwdriver

4" x 4" Block of Wood

1. Tip table slightly and place a 4" x 4" block underneath base as shown. CAUTION: At least 2 people are required for this step.



2. Remove the two screws that secure the electrical plate to the table base.



3. Remove three wires connected to the power inlet.



4. Remove the two screws that secure the foot control connector to the table base.

5. Remove electrical plate from table and discard.



6. Attach foot control connector to new electrical plate.

7. Reattach three wires to new power inlet. Be sure to orientate wires as shown.



8. Attach new plate to table base.

9. Carefully remove 4" x 4" wood block and set table gently on floor.

