


**⚠ DANGER**

Always disconnect the power cord from the wall outlet before performing service on the table to prevent electrical shock. Failure to do so could result in severe personal injury or death.

**⚠ WARNING**

Refer to the Operator's Manual supplied with the unit for complete instructions on operation. Failure to do so could result in personal injury.

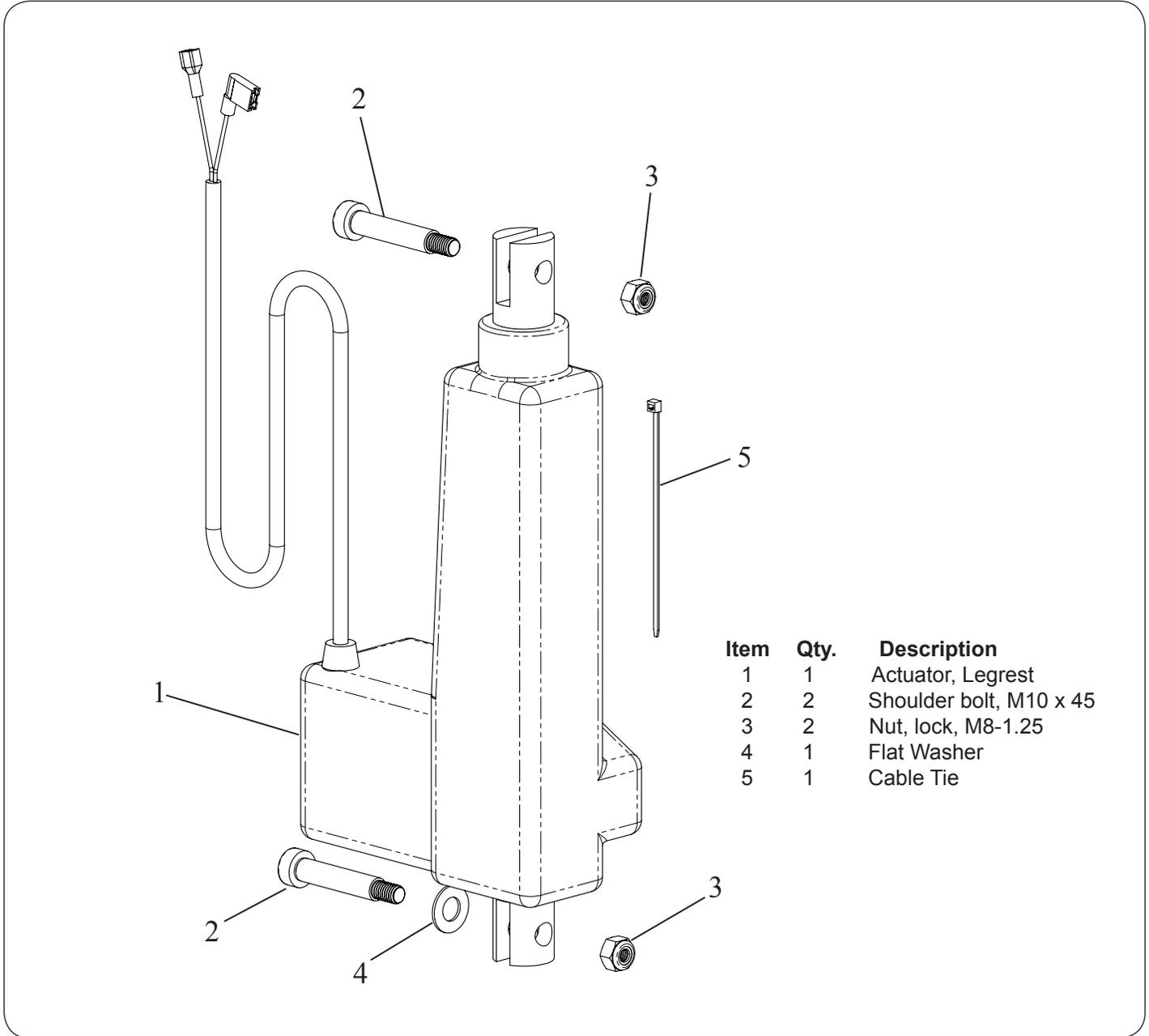

Item	Qty.	Description
1	1	Actuator, Legrest
2	2	Shoulder bolt, M10 x 45
3	2	Nut, lock, M8-1.25
4	1	Flat Washer
5	1	Cable Tie

13901 Main Street, Menomonee Falls, WI 53051  
 Phone: 1.888.BREWER.1 (1.888.273.9371)  
 Fax: 1.262.251.1786  
 Web: www.brewercompany.com

1. Plug the power cord into an electrical receptacle. Raise the table, tilt the seat back, and raise the leg section to gain access to the legrest. Unplug the power cord.
2. Mark the actuator mounting bolts locations. Remove the bottom mounting nut and bolt. Allow the actuator and legrest to swing down.



3. Remove the wire clamps from the actuator cable. Disconnect the wire from the limit switch. Make sure to note where each color wire was connected.



4. Remove the top mounting nut and bolt. Remove the actuator.
5. Install the actuator using the M10 shoulder bolt and nut supplied to the bottom mounting hole.
6. Attach the actuator to the top mounting hole using the M10 shoulder bolt and nut supplied.
7. Connect the wires and install the wire clamps.
8. Plug the power cord into an electrical receptacle and make sure the actuator functions properly.
9. Pull out the debris tray. The legrest should not move upward when attempting to raise the legrest with the control.

